#### **Product Data Sheet**

Propulse Ear Irrigator Mushroom Valve User Guide

# Are you using the right mushroom valve and washer for your Propulse Ear Irrigator?

#### **Mushroom Valves**



The mushroom valve on the left is for the Propulse G5 & PP17, and the one on the right is for the older model Propulse units (II, III & NG). As you can see, they differ in a couple of key areas:

- The G5 / PP17 mushroom valve has a longer central spoke
- The G5 / PP17 mushroom valve has wider outside spokes
- The G5 / PP17 mushroom valve is a brighter white in colour

## Washers / 'O-rings'



The washer on the left is for the Propulse G5 & PP17. The most obvious difference is that it is made of red rubber instead of black. It is also slightly thicker, and smaller in diameter.

### **Key Points:**

- When the mushroom valve is fitted correctly, it should lift slightly with the tension of your finger, when pressed.
- Be careful when changing the valves and washers, as any damaged spokes will cause the tank to leak.
- Be sure to check the valves and washers prior to each use, as they do wear through normal use and may need changing.



Mirage Health Group Ltd, 1 Little Mundells, Welwyn Garden City,

Hertfordshire AL7 1EW

Tel: +44 (0) 845 130 5440 Fax: +44 (0) 845 130 6440