NOT FOR PUBLIC RELEASE

Navina™ Mini

EN

INTENDED USE

Navina Mini is intended for transanal irrigation by installing a low volume of water into rectum through a single use rectal cone.

INDICATIONS FOR USE

Navina Mini is indicated for adults, adolescents and children from 3 years who suffer from incomplete emptying and/or fecal incontinence.

INTENDED CLINICAL BENEFIT

The treatment is intended to facilitate adequate emptying of the rectum and/or reduce accidental bowel leakage.

CONTRAINDICATIONS

Do NOT use Navina Mini if you have one or more of the following:

- Known anal stenosis
- Active inflammatory bowel disease
- Acute diverticulitis and diverticular abscess
- Colorectal cancer
- Ischemic colitis
- You are within three months of anal or colorectal surgery

Since the list may not be exhaustive, healthcare professionals should always consider individual users as well.

PRECAUTIONS

This product is NOT recommended for pregnant women.

Special care must be taken if you have or have had any of the following:

- Painful anorectal conditions - any condition which may cause pain or bleeding, e.g. anal fissure, anal fistula, third or fourth grade of hemorrhoids
- Known colorectal stenosis
- Previous endoscopic polypectomy within 4 weeks
- A risk of uncontrolled autonomic dysreflexia
- Severe diverticulosis
- Irradiation therapy in the abdominal or pelvic region within 3 months or until cessation of proctitis
- Previous major pelvic surgery, i.e. close monitoring is recommended during initiation of therapy
- Changed stool pattern such as sudden diarrhea of unknown cause.

Transanal irrigation can be associated with symptoms like bowel discomfort and light bleeding from the rectum. If any of the symptoms remain or increase, consult your healthcare professional for advice.

USAGE

- The Navina Mini cone is for single use only.
- Use only original Wellsept components. No modification of this product is allowed.
- Navina Mini shall only be used with water.
- Use only after assessment and training from a healthcare professional.
- Children shall be accompanied by an adult caregiver until the caregiver considers the child able to perform the procedure by themselves.
- Navina Mini shall be kept out of reach of small children.
- All the Navina Mini components shall be disposed in the household waste container.
- Any serious adverse reaction occurring when using the Navina Mini should be reported to the manufacturer and your local health authority.

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UK:
Wellsept HealthCare
Stroudhurst Business Park
Brunel Way
Stonehouse
Gloucestershire
GL10 3GB
Tel: +44 (0)1453 793228
Web: www.wellsept.co.uk

CH:
Wellsept AG
Täferweg 1
CH-5404 Baden-Dättwil
Telefon: +41 (0) 800 620 240
Fax: +41 (0) 800 620 241
Web: www.wellsept.ch
Web: fr.wellsept.ch

BELGIUM, THE NETHERLANDS AND LUXEMBOURG:
Wellsept HealthCare
Signalrood 55
NL-2718 SG Zoetermeer
The Netherlands
Tel NL: +31 79 360 10 50
Fax NL: +31 79 362 37 48
Tel BE: +32 3 232 81 50
Fax BE: +32 3 215 30 66
Dutch web: www.wellsept.nl

OTHER MARKETS:
For more information please go to
www.wellsept.com

CE

Manufacturer
Wellsept HealthCare,
Aminogatan 1, P.O. Box 145E-431 21
Mölnådal, Sweden

Wellsept®
A REAL DIFFERENCE

Navina™ Mini

EN

INSTRUCTIONS FOR USE:

Please read these instructions before use. The instructions should be seen as complementary to the advice given by your healthcare professional. Navina Mini is for prescription use only.

Preparation

1. Open the lid and fill the entire water container with lukewarm (36-39° C) tap water.
2. Close the lid.
3. If an extension tube is used: Assemble the extension tube with water.
4. Open the bag and assemble the cone to the water container or if used, the extension tube.
5. Activate the slippery coating of the cone to its base by wetting it with tap water.

Instillation

6. Sit on or stand over the toilet and insert the cone into rectum carefully.
7. When the cone is in place, gently squeeze the water container to instill the water. Make sure you only insert the pointed section of the cone and stop when you reach the wider base.
8. Pull out the cone and let the bowel empty.

Disassembling and cleaning

9. Put the cone back in the bag and dispose in the household waste.
10. If an extension tube is used: Disassemble the extension tube and water container.
11. If an extension tube is used: Clean the extension tube by rinsing with soapy lukewarm water only and let dry.
12. Open the lid of the water container and empty remaining water.
13. Gently clean the water container after each use with soapy lukewarm water only. Let dry.
14. Wash your hands.

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EN Navina Mini has a one-way valve in the lid, so the water container can be squeezed more than once, without air or water being sucked back into the water container during use. For this reason, it will stay compressed until the lid is opened. The rectal cone has a slippery hydrophilic surface for easy insertion and withdrawal. The components of Navina Mini are not manufactured with latex, PVC or phthalates.

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